



INSTRUCTIONS FOR USING LANCING DEVICE AND LANCET

The Lancing Device allows you to collect your capillary blood from fingertip (and alternative sites) easily and virtually pain free.

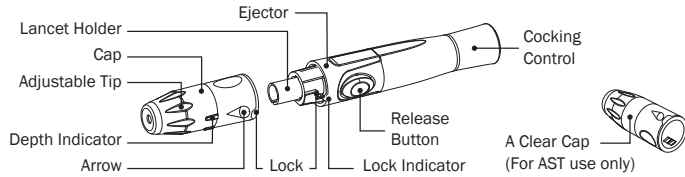

The Lancing Device offers you 6 depths of skin penetration:

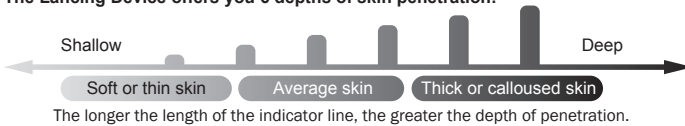


WARNING


To reduce the chance of infection:

- ▶ Never share a lancet or the lancing device.
- ▶ Always use a new, sterile lancet. Lancets are for single use only.
- ▶ Avoid getting hand lotion, oils, dirt or debris in or on the lancets and the lancing device.
- ▶ The used lancet may potentially be biohazardous. Discard it according to your local regulations.


Setting up the Lancing Device

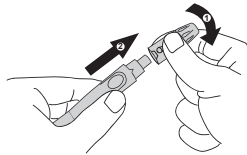


- 1.** Twist and pull off the cap of the lancing device.

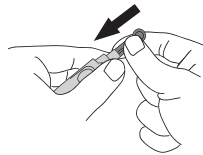


- 2.** Insert a lancet into the lancet holder and push down firmly until it is fully secured.

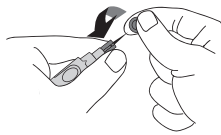


- 3.** Twist the protective disk off the lancet.

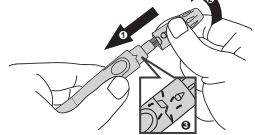

- 4.** Replace and twist the cap to close the lock as shown in the picture until you hear the click. When the lock is properly closed lock indicator is in the extreme right position.

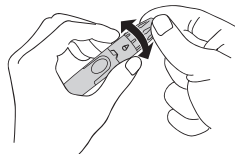

- 5.** Holding the cap select on the depth of penetration by turning the adjustable tip in either direction so that the arrow on the cap points to the desired depth.

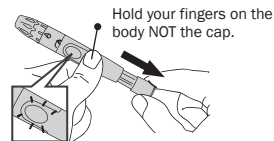


- 6.** Pull the cocking control back until it clicks. You will see a color change inside the release button when it is ready.
▶ **If it does not click, the device may have been cocked when the lancet was inserted.**

NOTE

Before collecting a drop of blood:


- ▶ Wash and dry your hands before you start.
- ▶ Select the puncture site at the fingertip.
- ▶ Rub the puncture site for about 20 seconds before penetration.
- ▶ Clean the site using cotton swab moistened with 70% alcohol and let it air dry.

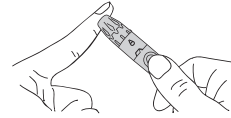

- 7.** Lancing the finger
Press the lancing device's tip firmly against the lower side of your fingertip. Press the release button to prick your finger, then a click indicates that the puncture is complete.



- 8.** Remove the device from the puncture site. After penetration, remove the first drop of blood with a clean cotton swab. Gently squeeze the punctured area to obtain another drop of blood. Be careful NOT to smear the blood sample. Apply the blood to the test strip.

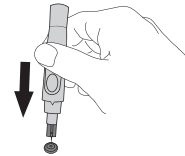


- 9.** Remove the lancet by twisting the cap off first. Safely dispose of the used lancet by placing the protective disk on a hard surface, and pushing the exposed tip into the disk.



- 10.** Slide the ejector forward to remove the used lancet. Dispose of the lancet according to your local regulations.
▶ **For collecting blood samples from sites other than the fingertip, use the clear cap.**

Alternative Site Testing (AST)

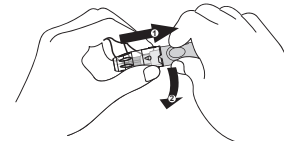

NOTE

This option is not available for all devices. Refer to the **Intended Use** in your meter's Owner's Manual if you can perform AST with your device.

Testing on sites other than the fingertip is referred to as alternative site testing (AST). Please read the **Alternative Site Testing** section in your Owner's Manual before you perform AST.

Substitute the lancing device cap with the clear cap for alternative site testing. Pull the cocking control back until it clicks.

When lancing the forearm, upper arm, hand, thigh, or calf, avoid lancing the areas with obvious veins in order to prevent excess bleeding.

Cleaning

The lancet is disposable and is intended for single use only. The lancing device is reusable.

To clean the lancing device, use a mild soap and water to wipe clean the lancing device. DO NOT rinse the device under water. Disinfect the cap by placing it in 70% alcohol for 10 minutes and then allow it to air dry. Do NOT place the device in a dishwasher or use detergents.

Symbol Information

Symbol	Referent
	Do not reuse (Lancet use only)
	Consult instructions for use
	Use by (Lancet use only)
	Manufacturer
	Authorized representative in the European Community
	Sterilized using irradiation (Lancet use only)
	Caution, consult accompanying documents (Lancet use only)
	Batch code
	CE mark

TaiDoc Technology Corporation
B1-7F, No. 127, Wugong 2nd Rd., Wugu Dist., 24888 New Taipei City, Taiwan

MedNet GmbH
Borkstraße 10, 48163 Münster, Germany

STERILE